

GENERAL

The Powerex medical vacuum system is designed to create a suction system to remove unwanted fluids or gases from hospital/ laboratory working areas. The medical vacuum system package is compliant with the NFPA 99 requirements for Risk Category 1 systems. Each system is completely tested before shipment and includes:

- Multiple vacuum pumps and associated equipment
- AMSE air receiver
- Medical control panel

Each pump is factory piped to a common intake manifold. Vibration isolation pads are included with the system.

ROTARY VANE VACUUM PUMP

The medical vacuum pumps shall be of the rotary vane air-cooled design. Each vacuum pump shall be direct-driven through a shaft coupling by a TEFC electric motor. Belt drives shall not be permitted. The vacuum pumps shall be mounted on vibration isolators.

- Each vacuum pump shall be air-cooled and have absolutely no water requirements.
- Each pump is equipped with a 10 micron inlet filter for removal of particulates.
- Lubrication shall be provided by an integral, fully re-circulating oil supply that is filtered by an automotive-type, spin-on oil filter. Non-re-circulating (once-through) or partial re-circulating oil supply systems shall not be permitted. Each vacuum pump shall be capable of operation with ISO 100 viscosity oil.
- The oil separation systems shall be integral and shall consist of no less than three stages of internally installed oil and smoke eliminators through which the exhaust gas stream must pass. This system shall consist of bulk separation, oil mist elimination, and smoke elimination, and shall be capable of removing 99.9+ percent of all oil and smoke particles from the exhaust gas stream.
- Each vacuum pump shall include a built-in, anti-suck-back valve, mounted at the pump inlet, and three sliding vanes.

MOTOR

The motor is continuous duty, C-face, TEFC, suitable for 208-230, or 460V, 3 phase, 60 hertz electrical operation.

AIR RECEIVER

The system shall include an ASME rated air receiver. The tank shall be equipped with a vacuum gauge, a sight gauge, by-pass valves, and a manual drain.

STANDARD HMI CONTROL PANEL

The control system provides automatic lead/lag sequencing and automatic alternation of all pumps in order to equalize the amount of usage among the available vacuum pumps. The HMI (Human Machine Interface) control system shall include:

- PLC controller and a color touch screen HMI (Human Machine Interface) panel which displays the operating status of the unit.
- UL508A listed control panel in a NEMA 12 enclosure. The panel door will include: the HMI touch screen, an audible and visual alarms with an acknowledge button, and an HOA switch for each pump
- Magnetic starters
- Vacuum transducer for process control
- Single point power connection
- Redundant 120Vac control transformers with fused primary and secondary protection
- System overload trip, high temperature conditions or maintenance intervals for the pump will result in visual and audible alarms.

OPTIONAL PBMI CONTROL PANEL

The PBMI control panel shall include all features of the standard HMI control panel with the addition of the gateway server card. The PBMI control system shall include:

- Building automation communication gateway with BacNet[®] protocol and Web server features. Web server s features include email notifications in case the system is in alarm or has achieved one its maintenance intervals and requires service.
- Ethernet port for connection to BacNet[®] server or direct connection to facility Ethernet for viewing of system operations and status via device IP-address

OPTIONAL BASIC CONTROL PANEL

The control system provides automatic lead/lag sequencing and automatic alternation of all compressors in order to equalize the amount of usage among the available vacuum pumps. The Basic control system shall include:

- UL508A listed control panel in a NEMA 12



**Lubricated Vane Medical
Vacuum System**
Rev. 10/3/13

enclosure with the following accessories for each pump: Externally operable disconnect, magnetic starter with 3-leg overload protection, Hand/Off/Auto lighted selector switch, minimum run timer to prevent short cycling of the pump, and hour meter. Standard features shall also include:

- Main power on light, timed lead/lag pump alternation, a reserve pump in-use alarm with visual and audible indications, and redundant control circuit transformers with fused primary and secondary protection.
- Dry contacts on a labeled terminal strip for remote alarm monitoring and an acknowledge pushbutton for horn silencing.
- Control logic to start the lag pump automatically if the lead vacuum pump fails to operate.
- A high discharge air temperature shutdown alarm with visual and audible indicators and dry contacts

AVAILABLE OPTIONS




- ☐ Oil temperature gauges
- ☐ Internal tank lining for corrosion resistance



Medical Vacuum Tankmount 1.5-2HP with HMI/PBMI

Rev. 7/25/12

| MODEL | DIM. A | DIM. B | DIM. C | Inlet | Outlet |
|----------|-----------|-----------|-----------|--------|--------|
| VVTD0153 | 54" | 34" | 85" | 1-1/2" | 1-1/2" |
| VVTD0203 | 54" | 34" | 85" | 1-1/2" | 1-1/2" |

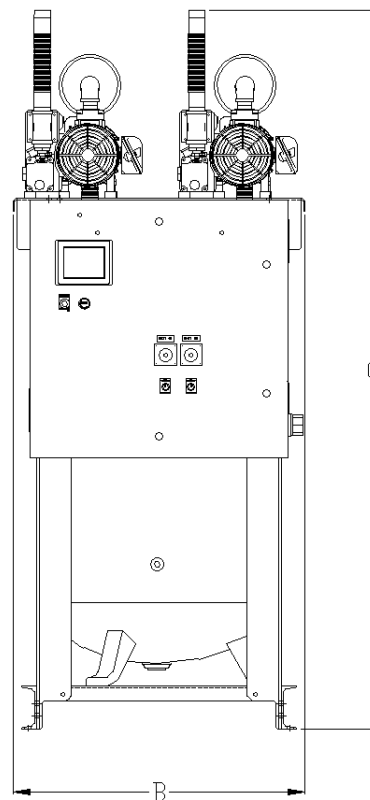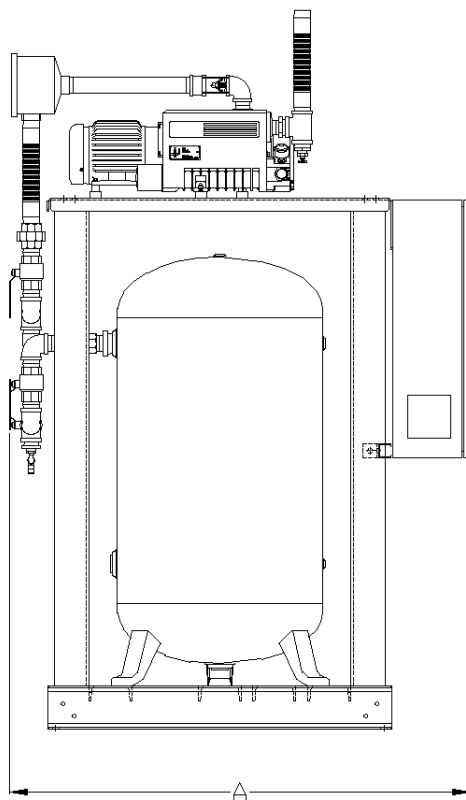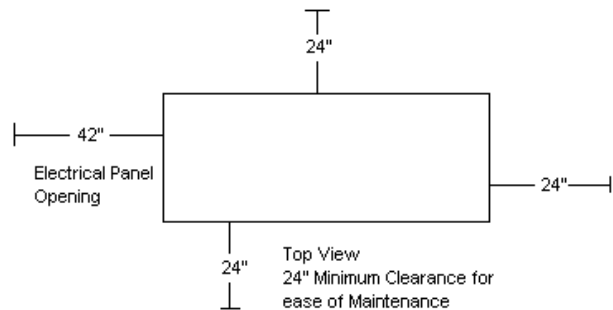

| Medical Vacuum Tankmount with HMI/PBMI | | | | | | | | | | |
|--|-----|------------------|---|------------------------|---------------------|-----------------------------|---------------|------|------|----------------------|
| MODEL | HP | SCFM @ 19" Hg | NFPA System Capacity ¹ | TANK SIZE (Gal.) | BTU/HR ² | dB(A) ³ LEVEL | SYSTEM F.L.A. | | | SYSTEM WT. (LBS.) |
| | | | | | | | 208V | 230V | 460V | |
| VVTD0153 | 1.5 | 7 | 7 | 80V | 3,818 | 70 | 11 | 10.8 | 6.4 | 1,200 |
| VVTD0203 | 2 | 11 | 11 | 80V | 5,090 | 70 | 13.2 | 12.8 | 7.4 | 1,300 |




- Notes:
- 1- System Capacity is shown with one or more pumps in reserve per NFPA 99
 - 2 - BTU/HR Levels are shown with reserve pump(s) on standby
 - 3 - dB(A) is shown with one pump in reserve per NFPA 99
 - 4 - 3 Year Limited Warranty



Medical Vacuum Tankmount 3-5HP with HMI/PBMI

Rev. 7/25/12

| MODEL | DIM. A | DIM. B | DIM. C | Inlet | Outlet |
|----------|-----------|-----------|-----------|--------|--------|
| VVTD0303 | 54" | 34" | 85" | 1-1/2" | 1-1/2" |
| VVTD0403 | 54" | 34" | 85" | 1-1/2" | 1-1/2" |
| VVTD0503 | 58" | 42" | 89" | 2" | 2" |

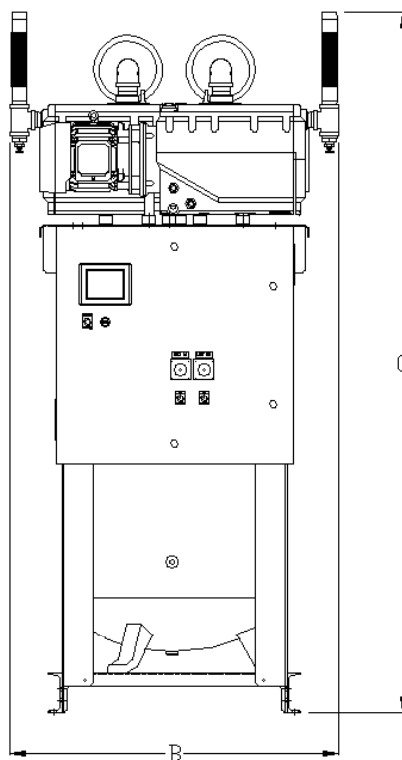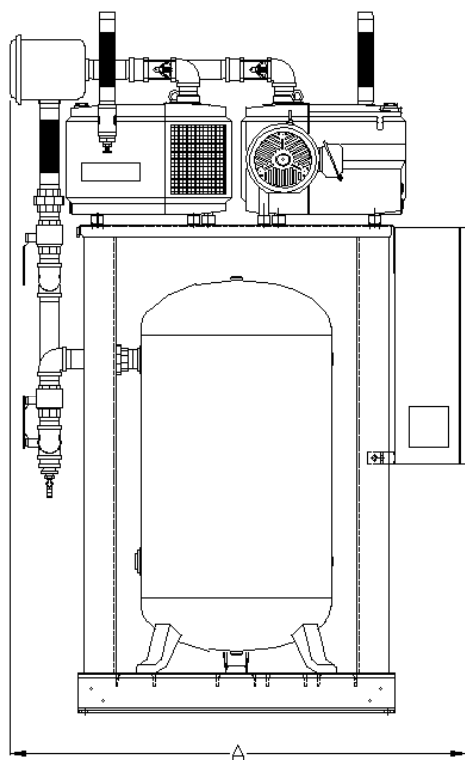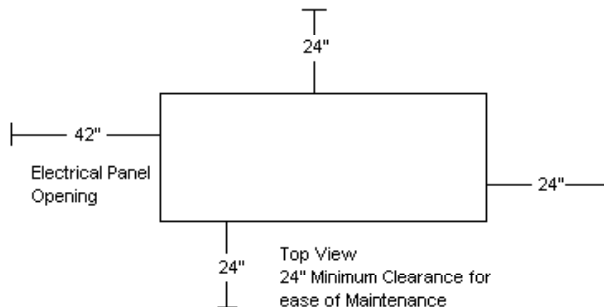

| Medical Vacuum Tankmount with HMI/PBMI | | | | | | | | | | |
|--|----|------------------|---|------------------------|---------------------|-----------------------------|---------------|------|------|----------------------|
| MODEL | HP | SCFM @ 19" Hg | NFPA System Capacity ¹ | TANK SIZE (Gal.) | BTU/HR ² | dB(A) ³ LEVEL | SYSTEM F.L.A. | | | SYSTEM WT. (LBS.) |
| | | | | | | | 208V | 230V | 460V | |
| VVTD0303 | 3 | 17 | 17 | 80V | 7,635 | 70 | 18.8 | 17.6 | 9.8 | 1,400 |
| VVTD0403 | 5 | 26 | 26 | 80V | 12,725 | 71 | 29.6 | 28 | 15 | 1,500 |
| VVTD0503 | 5 | 41 | 41 | 80V | 12,725 | 74 | 30.4 | 28 | 15 | 1,525 |




- Notes:
- 1- System Capacity is shown with one or more pumps in reserve per NFPA 99
 - 2 - BTU/HR Levels are shown with reserve pump(s) on standby
 - 3 - dB(A) is shown with one pump in reserve per NFPA 99
 - 4 - 3 Year Limited Warranty



Medical Vacuum Tankmount with HMI/PBMI

Rev. 7/25/12

| MODEL | DIM. A | DIM. B | DIM. C | Inlet | Outlet |
|---------|-----------|-----------|-----------|--------|--------|
| VTD0153 | 71" | 29" | 68" | 1-1/2" | 1-1/2" |
| VTD0203 | 70" | 29" | 68" | 1-1/2" | 1-1/2" |
| VTD0303 | 70" | 31" | 68" | 1-1/2" | 1-1/2" |
| VTD0404 | 77" | 32" | 72" | 1-1/2" | 1-1/2" |
| VTD0504 | 77" | 38" | 71" | 2" | 1-1/2" |
| VTD0755 | 85" | 44" | 74" | 2" | 2" |
| VTD1006 | 90" | 50" | 75" | 2" | 2" |

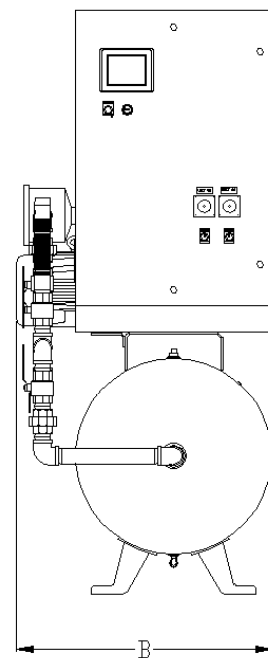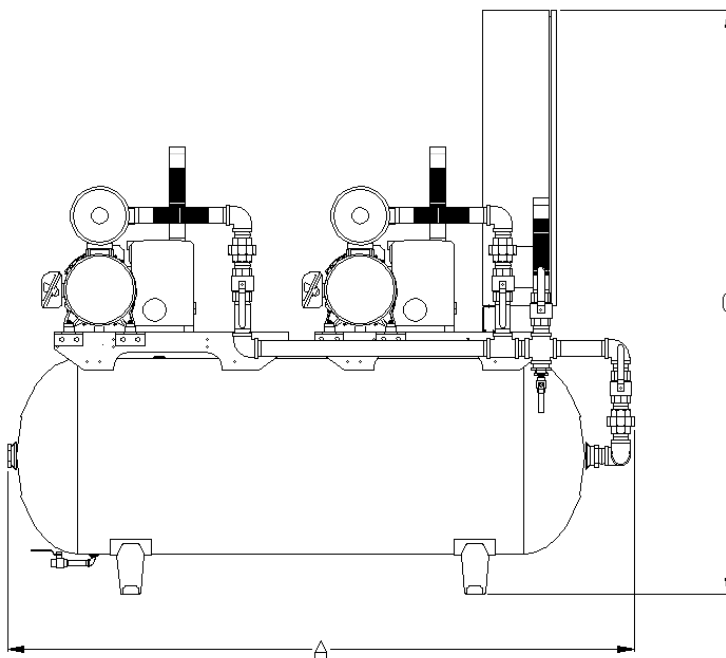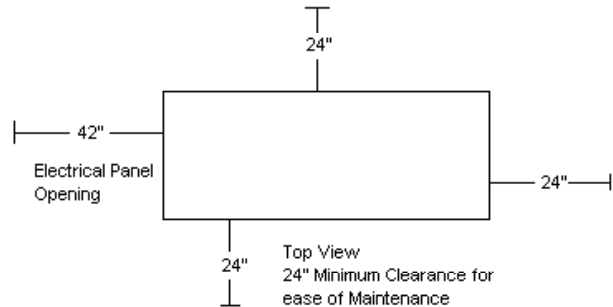

| Medical Vacuum Tankmount with HMI/PBMI | | | | | | | | | | |
|--|-----|--------------------------|---|------------------------|---------------------|-----------------------------|---------------|------|------|----------------------|
| MODEL | HP | SCFM @ 19" Hg Each | NFPA System Capacity ¹ | TANK SIZE (Gal.) | BTU/HR ² | dB(A) ³ LEVEL | SYSTEM F.L.A. | | | SYSTEM WT. (LBS.) |
| | | | | | | | 208V | 230V | 460V | |
| VTD0153 | 1.5 | 7 | 7 | 80 | 3,818 | 70 | 11 | 10.8 | 6.4 | 850 |
| VTD0203 | 2 | 11 | 11 | 80 | 5,090 | 70 | 13.2 | 12.8 | 7.4 | 950 |
| VTD0303 | 3 | 17 | 17 | 80 | 7,635 | 70 | 18.8 | 17.6 | 9.8 | 1,100 |
| VTD0404 | 5 | 26 | 26 | 120 | 12,725 | 71 | 29.6 | 28 | 15 | 1,250 |
| VTD0504 | 5 | 41 | 41 | 120 | 12,725 | 74 | 30.4 | 28 | 15 | 1,250 |
| VTD0755 | 7.5 | 52 | 52 | 200 | 19,088 | 79 | 44 | 42.8 | 22.4 | 2,200 |
| VTD1006 | 10 | 77 | 77 | 240 | 25,450 | 81 | 54.8 | 53.2 | 27.6 | 2,300 |

- Notes:
- 1- System Capacity is shown with one or more pumps in reserve per NFPA 99
 - 2 - BTU/HR Levels are shown with reserve pump(s) on standby
 - 3 - dB(A) is shown with one pump in reserve per NFPA 99
 - 4 - 3 Year Limited Warranty